


COMPONENTS

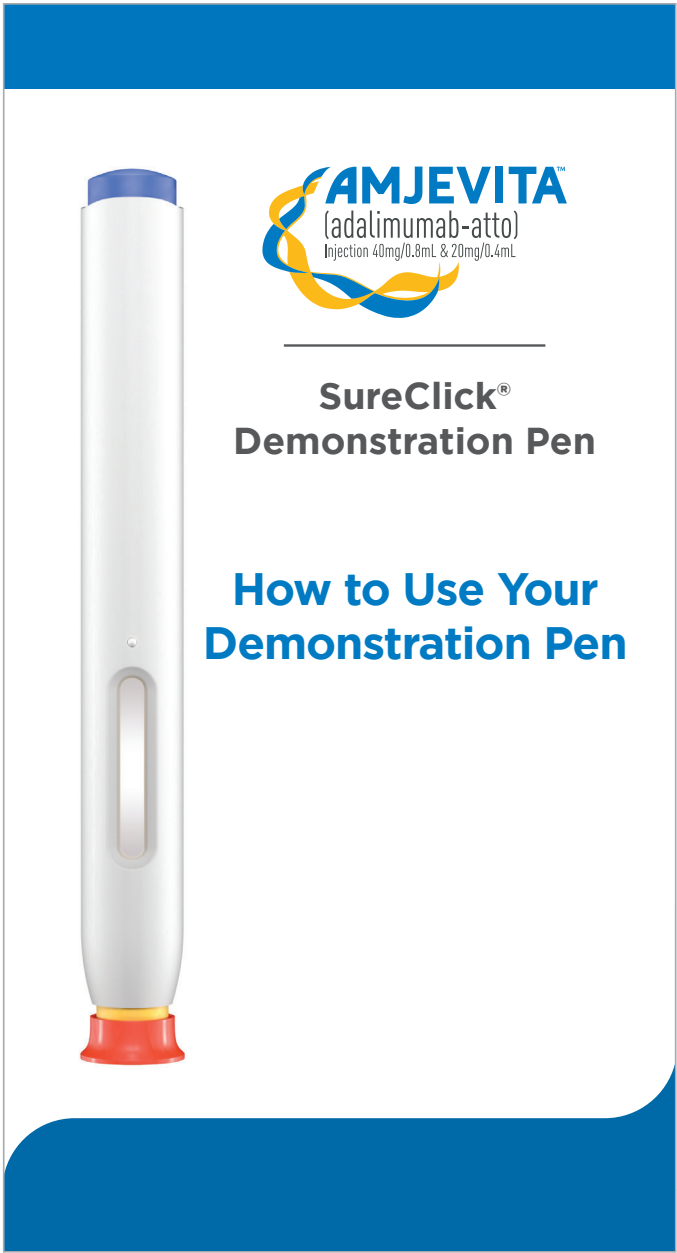

Demo Instructional Guide




IFU



PI/MG

Inlay with Reset Tool

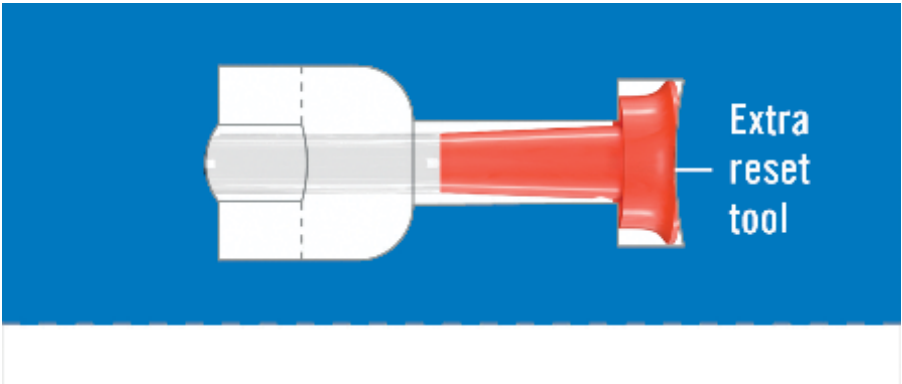

Practice builds confidence

A demonstration pen and guide have been provided to help you build confidence as you practice the injection steps.

Practice as many times as you need to and you will be self-injecting in no time.

Uses for your demonstration pen:

- Helping a caregiver to practice completing your injection
- Trying out new injection site(s)
- Practicing your key steps before you inject
- Practicing the steps if you have not injected in a while

Need more help?

If you need support, contact your healthcare provider or call **1-888-8AMJEVITA** (1-888-826-5384), 8AM-8PM ET, Monday to Friday or visit **AMJEVITA.com**.

To see a demonstration of how to perform your self-injection, you can access the SureClick® Autoinjector overview video below.



Hold your smartphone camera over the QR code to access the overview video.

Or access the video directly at: **amjevita.com/supportplus/how-to-take-amjevita**



Please see accompanying AMJEVITA™ full Prescribing Information, including Medication Guide and Instructions for Use.



© 2023 Amgen Inc.
All rights reserved. Not for Reproduction.
USA-501-80020



SureClick® Demonstration Pen

How to Use Your Demonstration Pen

Getting familiar with self-injecting

Are you new to using the SureClick® Autoinjector? We're here to support you.

Self-injecting may feel unfamiliar. It may leave you with questions. It's OK. You are not alone.

Follow the enclosed guide for supplemental information on using your demonstration pen with AMJEVITA™.


Prior to administering your medication, thoroughly review the Instructions for Use provided with your medication. The Instructions for Use contain everything you need to know about how to use AMJEVITA™.




Using your SureClick® Demonstration Pen

Prep

SELECT
an injection
site.

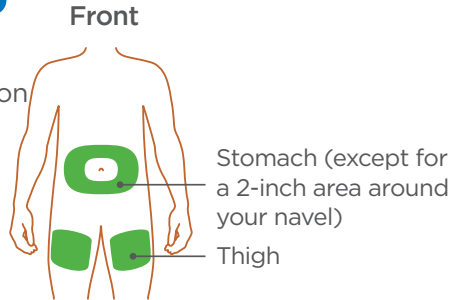


REMOVE the cap.

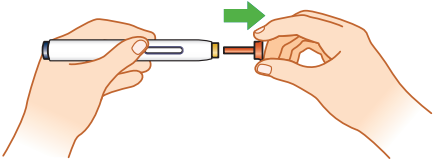


PINCH or **STRETCH** the skin to create a firm surface.

Pinch
(if using the stomach)

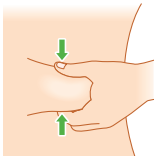

Stretch
(if using the thigh)

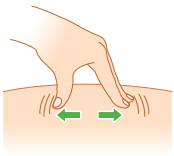


PLACE the safety guard on the skin and be sure you can see the entire medication window.

Inject


PUSH

down until the yellow safety guard stops moving and **HOLD** until injection completes.

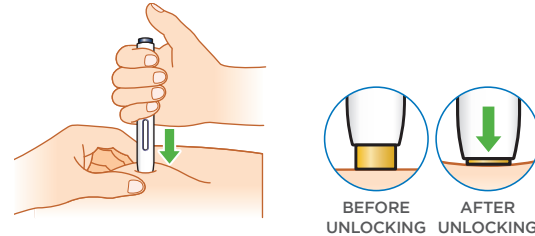


PRESS

the start button; you will hear a CLICK. KEEP HOLDING the Autoinjector down.

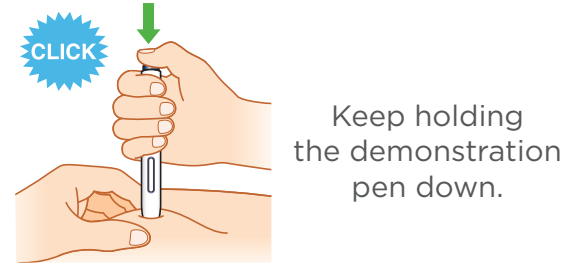


WATCH & CONFIRM

window is fully yellow.

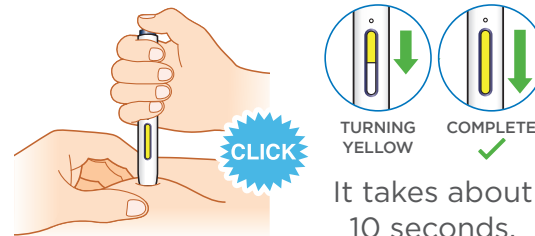

Reset

PUSH the reset tool completely back into the demonstration pen. You will hear a click.

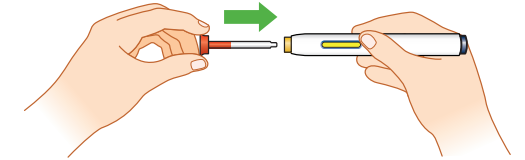

Tips for success

- ✓ Keep the skin pinched or stretched throughout the entire injection.
- ✓ Make sure the safety guard is pushed all the way down before pressing the start button.
- ✓ Make sure you get the full injection by confirming the medication window is yellow before removing the pen from your skin.
- ✓ The colors of the demonstration pen may differ from your SureClick® Autoinjector.
- ✓ Only recap your demonstration pen. **DO NOT** recap your Autoinjector.