


COMPONENTS

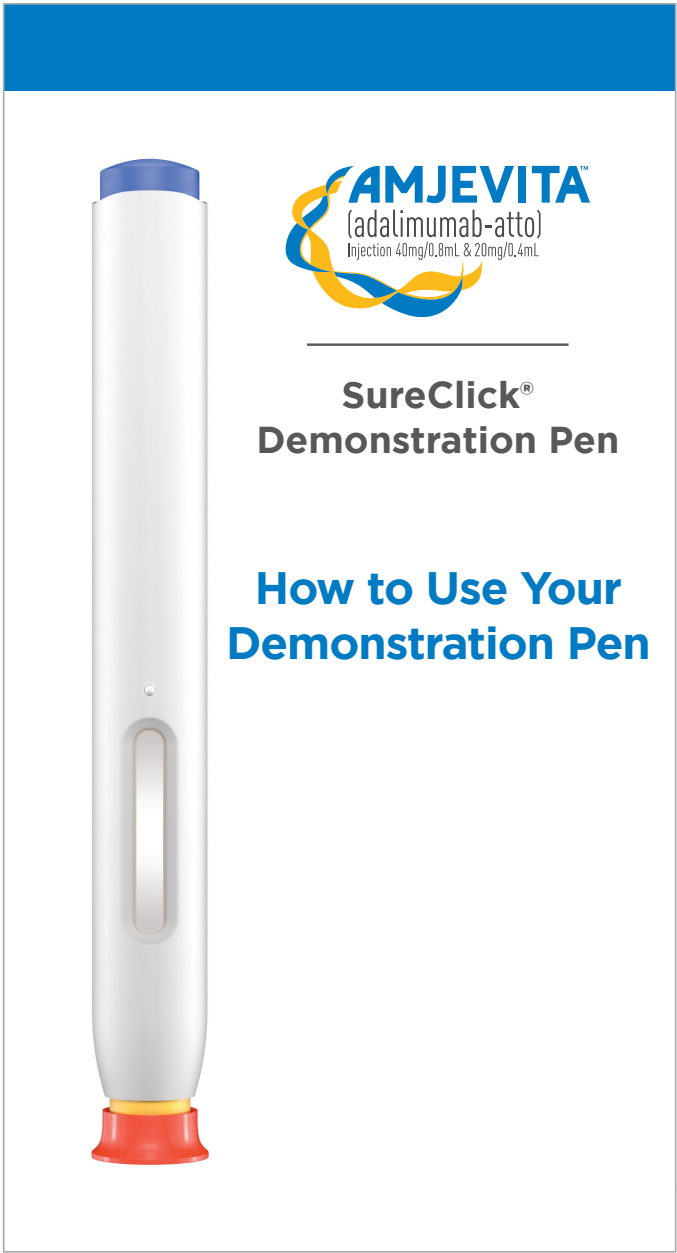

Demo Instructional Guide



IFU



PI/MG



Magnet

## Practice builds confidence

A demonstration pen and guide have been provided to help you build confidence as you practice the injection steps.

**Practice as many times as you need to and you will be self-injecting in no time.**



### Uses for your demonstration pen:

- Helping a caregiver to practice completing your injection
- Trying out new injection site(s)
- Practicing your key steps before you inject
- Practicing the steps if you have not injected in a while

## Need more help?

If you need support, contact your healthcare provider or call **1-888-8AMJEVITA** (1-888-826-5384) 8AM-8PM ET, Monday to Friday or visit **AMJEVITA.com**.

To see a demonstration of how to perform your self-injection, you can access the SureClick® Autoinjector overview video below.



Hold your smartphone camera over the QR code to access the overview video.

Or access the video directly at: [amjevita.com/supportplus/how-to-take-amjevita](https://amjevita.com/supportplus/how-to-take-amjevita)



**Please see accompanying AMJEVITA™ full Prescribing Information, including Medication Guide and Instructions for Use.**



© 2023 Amgen Inc.  
All rights reserved. Not for Reproduction.  
USA-501-80024



### SureClick® Demonstration Pen

## How to Use Your Demonstration Pen

## Getting familiar with self-injecting

**Are you new to using the SureClick® Autoinjector?** We're here to support you.

Self-injecting may feel unfamiliar. It may leave you with questions. It's OK. You are not alone.


Follow the enclosed guide for supplemental information on using your demonstration pen with AMJEVITA™.

**Prior to administering your medication, thoroughly review the Instructions for Use provided with your medication. The Instructions for Use contain everything you need to know about how to use AMJEVITA™.**




# Using your SureClick® Demonstration Pen

## Prep

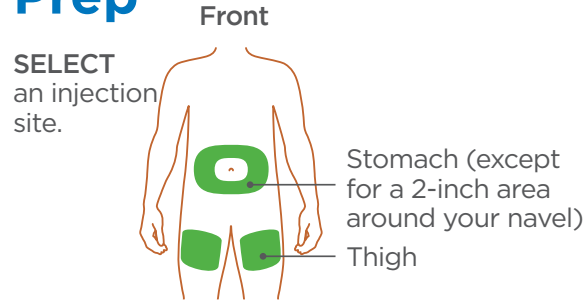


**REMOVE** the cap.

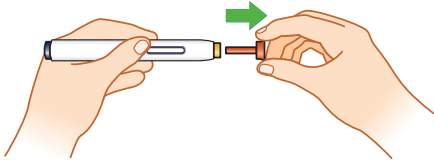


**PINCH** or **STRETCH** the skin to create a firm surface.

Pinch (if using the stomach)

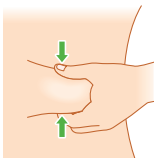

Stretch (if using the thigh)

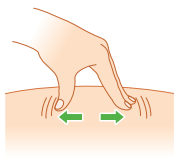


**PLACE** the safety guard on the skin and be sure you can see the entire medication window.

## Inject

### PUSH

**PUSH** down until the yellow safety guard stops moving and **HOLD** until injection completes.

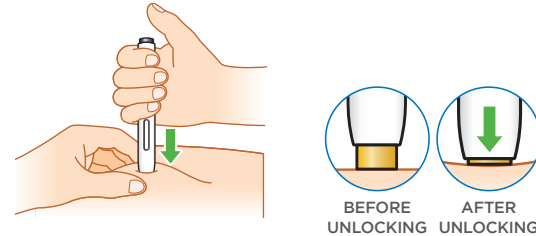


### PRESS

**PRESS** the start button; you will hear a **CLICK**. **KEEP HOLDING** the Autoinjector down.




**WATCH & CONFIRM** window is fully yellow.

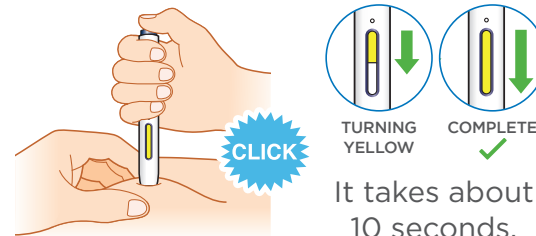

## Reset

**PUSH** the reset tool completely back into the demonstration pen. You will hear a click.

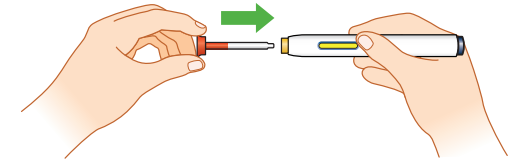

## Tips for success

- ✓ Make sure you can identify the start button from the injection end.
- ✓ Keep the skin pinched or stretched throughout the entire injection.
- ✓ Make sure the safety guard is pushed all the way down before pressing the start button.
- ✓ Make sure you get the full injection by confirming the medication window is yellow before removing the pen from your skin.
- ✓ The colors of the demonstration pen may differ from your SureClick® Autoinjector.
- ✓ Only recap your demonstration pen. **DO NOT** recap your Autoinjector.

# NEED HELP? HAVE QUESTIONS?

SUPPORT FOR AMJEVITA™ (adalimumab-atto) IS JUST A CLICK OR CALL AWAY

Additional  
Resources



[AMJEVITA.com](https://www.amgen.com/AMJEVITA)

Injection  
Overview Video



Hold smartphone  
camera over  
QR code

Live Support



Call 1-888-8AMJEVITA  
(1-888-826-5384)  
8AM-8PM ET, Monday  
to Friday



© 2023 Amgen Inc.

All rights reserved. Not for Reproduction. USA-501-80024